



# **GETTING STARTED**

Welcome to Jazz Imaging and thank you for choosing Jazz sensors! We hope you will have an outstanding experience with our products and services.

Everything you need to install your Jazz sensor is provided on the **USB key** shipped in your Jazz sensor box. Additional information can be found on our website.


http://www.jazzimaging.com/training

If you need assistance installing your Jazz sensor, you can schedule a convenient time with Jazz customer support team member through our website.

http://www.jazzimaging.com/calendar

For other questions or concerns, feel free to contact us at (567) 234-5299 or email support@jazzimaging.com

Your Jazz Customer Support Team

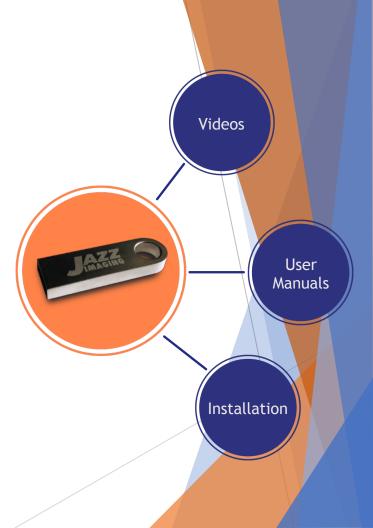




# STEP 1: INSTALL JAZZ CONTROL CENTER

Jazz Control Center (JCC) must be installed on each PC where images will be acquired.

- Insert the USB key into an available USB port on your computer.
- If the Jazz AutoStart Menu does not open automatically:
  - Open File Explorer
  - Locate Jazz USB drive (usually D: or E:)
  - Double click JazzAutoStart.exe to launch it.
- In the Jazz AutoStart Menu, select
  Jazz Control Center Installation under Jazz Control
  Center Software and follow the installation prompts.
- 4. Restart your computer when prompted.

#### After restarting:

- A Jazz Control Center shortcut will appear on the desktop.
- Jazz Control Center will start automatically.
- A mini Jazz Control Center icon will appear in the system tray (bottom-right corner of Windows).

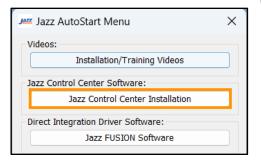






## **STEP 2: SENSOR SETUP**

- Jazz sensors come pre-licensed and ready to use. Simply connect the sensor to your PC, and a notification will appear confirming the sensor has been detected.
- ▶ If you don't see a notification, ensure that the USB-C connecter is securely plugged into the port at the end of the sensor.
- To verify the sensor and license status:
  - Navigate to the JCC Home Screen by clicking the JCC icon in the Windows systems tray or desktop.
  - 2. Check the sensor and license details displayed on the Home Screen.

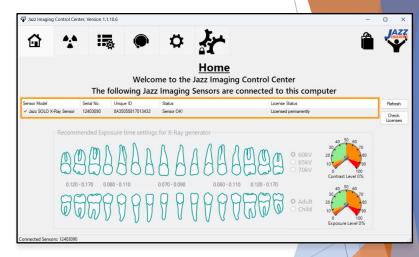

Jazz Control Center Home Screen



# **IMAGING SOFTWARES**

Need Help? Contact Jazz Support at (567) 234-5299





EzDent-i™



PLANMECA





Opendental software



...or other TWAIN



# YOU'RE DONE!

Start taking images with TWAIN

If you use:



**DENTRIX** ASCEND

APTERYX

**MOVERJET** 



# YOU'RE DONE!

Your sensor integrates directly

If you use:









 $CLINIVIEW^{\text{m}}$ 



SIDEXIS.



**GO TO PAGE 5** 

For something new, try:







**GO TO PAGE 6** 



## **STEP 3: INSTALL FUSION**

Close all your existing practice management and imaging software prior to installing any Jazz FUSION Software.

#### Open Jazz AutoStart:

Insert the USB key into your computer and launch JazzAutoStart.exe from the USB drive.

#### Select FUSION For Your Software:

- In Jazz AutoStart, click on Jazz FUSION Software
- Choose the version of FUSION that matches your imaging software.







#### Install FUSION:

Follow the on-screen prompts to compete the installation.

#### Start Using Your Jazz Sensor:

Once FUSION is installed, your Jazz sensor will work with your current imaging software, just like your original sensor.

#### Follow these steps if you have one of these Imaging Software applications:

















# USE JAZZ SENSORS WITH JAZZ CLASSIC FOR FREE

#### 1. Order CLASSIC:

- Visit http://www.jazzimaging.com/classic and log into your Jazz account.
- Complete the checkout process and access the download page.

#### 2. Install CLASSIC:

- Download CLASSIC and follow the CLASSIC Installation Guide to install it on your PC.
- Need help? Contact Jazz Support for installation assistance.

#### 3. Verify Your Practice Information:

 Open CLASSIC and confirm your Practice Name and Email Address.

#### 4. Activate Your Classic License:

- Connect your Jazz sensor to your PC.
- Take an image in CLASSIC to activate your 30-day license.
- Your license will automatically renew each time you capture an image using a licensed Jazz sensor.





(567) 234-5299 | WWW.JAZZIMAGING.COM

770 CHARCOT AVE SAN JOSE, CA 95131

Appointments: http://www.jazzimaging.com/calendar

Training/Support: http://www.jazzimaging.com/training

Classic Help Center: https://support.jazzimaging.com/

Email: support@jazzimaging.com